RE-USABLE ELECTRODES  
WIEDERVERWENDBARE ELEKTRODEN  
COUSSINETS D'ELECTRODE REUTILISABLES  
ELECTRODOS REUTILIZABLES  
50 x 50 mm (2 x 2 inch)

REF E-CM5050

- For single patient use only •Pour usage unique.
- Zur Verwendung durch nur eine Patientin •Para uso en un solo paciente

Manufactured by  
Hergestellt von:  
Fabriqué par:  
Fabricado por:

TensCare Ltd  
9 Blenheim Road  
Epsom, Surrey KT19 9BE UK  
Tel +44(0) 1372 723434  
painaway@tenscare.co.uk  
www.tenscare.co.uk

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#### Application of electrodes

1. Make sure stimulation device is switched off prior to applying or removing electrodes.
2. Clean electrode application area with soap and water, then rinse and dry.
3. Caution: Only apply electrodes to unbroken skin.
4. With electrodes still on the liner connect lead wires to the electrodes.
5. Remove an electrode from its liner ONLY by holding one edge and peeling it off the liner. Caution: Do not pull on the pig-tail wire. Note: Retain liner(s) for storage of electrodes after use.
6. Place electrode on skin by applying the centre of the electrode first then smoothing outwards to the edge. Use the illustrations in the instruction manual as a guide to positioning or consult with your medical adviser.
7. Attach the lead wires to the stimulation device.

#### Removal of electrodes

1. Turn off stimulation device and disconnect wires from it.
2. Remove electrodes from skin by peeling from an edge.

#### Storage and Maintenance

1. Between uses keep the electrodes on the liners and store in the re-sealable bag in a cool place, out of direct sunlight.
2. Note: The life of the electrode depends on skin condition, preparation, storage and climate.
3. When the electrode gel appears dry apply a few drops of water, spread with fingertip and let rest until gel regains its tackiness.
4. If electrode becomes over-saturated, allow to air dry in a refrigerator, adhesive side up, until gel regains its tackiness.

**CAUTION:** If skin irritation develops, discontinue use and consult your physician.

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