RE-USABLE ELECTRODES
WIEDERVERWENDBARE ELEKTRODEN
COUSSINETS D'ÉLECTRODE RÉUTILISABLES
ELECTRODOS REUTILIZABLES
50 x 90 mm (2 x 3.6 inch)

REF E-CM5090

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Manufactured by:
 Hergestellt von:
 Fabriqué par:
 Fabricado por:

TensCare Ltd
 9 Blenheim Road
 Epsom, Surrey KT19 9BE UK
 Tel +44(0) 1372 723434
 painaway@tenscare.co.uk
 www.tenscare.co.uk

CAUTION: If skin irritation develops, discontinue use and consult your physician.

1. Between uses keep the electrodes on the liners and store in the re-sealable bag in a cool place, out of direct sunlight.
2. Note: The life of the electrode depends on skin condition, preparation, storage and climate.
3. When the electrode gel appears dry apply a few drops of water, spread with fingertip and let rest until gel regains its tackiness.
4. If electrode becomes over-saturated, allow to air dry in a refrigerator, adhesive side up, until gel regains its tackiness.

Storage and Maintenance

1. Turn off stimulation device and disconnect wires from it.
2. Remove electrodes from skin by peeling from an edge.

Removal of electrodes

1. Make sure stimulation device is switched off prior to applying or removing electrodes.
2. Clean electrode application area with soap and water, then rinse and dry.
3. Caution: Only apply electrodes to unbroken skin.
4. With electrodes still on the liner connect lead wires to the electrodes.
5. Remove an electrode from its liner ONLY by holding one edge and peeling it off the liner. Caution: Do not pull on the pig-tail wire. Note: Retain liner(s) for storage of electrodes after use.
6. Place electrode on skin by applying the centre of the electrode first then smoothing outwards to the edge. Use the illustrations in the instruction manual as a guide to positioning or consult your medical adviser.
7. Attach the lead wires to the stimulation device.

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