

Go Gel

Water-based personal lubricant



Go Gel

Water-based personal lubricant

for use with
pelvic floor exercisers
and
for enhanced sexual comfort

Go Gel

is a sterile, water based gel which acts as a gentle lubricant
for a natural comfortable feeling
it is not greasy nor tacky and is can be used with latex condoms

- Water based, greaseless, transparent, and non-irritating
- Recommended for gynaecological, surgical lubrication and additional vaginal lubrication.,
- Suitable for electro-therapy and cytoscopic work and for instrument and digital examinations
- Spreads readily, adheres well
- Harmless to human tissue, does not injure rubber or metal appliances, instruments, natural, or synthetic textiles

Directions: Apply and spread onto the skin, condom or vaginal trainer.

Ingredients: Water, PEG-200. Methylparaben, Propylparaben, Sodium Hydroxide, Glycerine, Hydroxypropyl Methylcellulose

TensCare Stock Code: K-GO

Warnings

- If irritation occurs, discontinue use and consult your medical adviser.
- GO Gel is not a contraceptive and does not contain spermicide.
- Store at room temperature
- Keep out of reach of children

Distributed by

Tenscare Limited, 9 Blenheim Road, Longmead Business Park,
Epsom, Surrey, KT19 9BE, United Kingdom

Tel: +44 1372 72 34 34, www.tenscare.co.uk

TensCare™