## 4 Reusable electrode pads for electrotherapy unit

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TensCare™

### Instructions for use

For the desired success of any stimulation (TENS/EMS), it is important that electrode pads are placed sensibly. Leave 5-15 cm between the 2 electrode pads on a channel.

**Note:** The electrode pads must always be used in pairs (two electrode pads on each channel), so that the signal can flow in a circuit.

1) Before use make sure your skin is clean and dry. You can use conductivity barrier wipes for effective electrode pads contact such as TensCare Contact Skin Wipes (see E-CW50).

**Do not place electrode pads** on skin, which does not have normal sensation or on broken skin.

2) Push the pin ends of the lead wires firmly into the pigtail ends of the electrode pads.

3) Peel the electrode pads from their protective plastic shield by holding and lifting one corner of the pad and pulling.

**Do not pull** on the pigtail wire of the pad.  
**Always check** that the unit is **OFF** before applying or removing electrode pads.

4) Apply the adhesive side of the electrode pads around the area of pain and firmly press it to the skin. See examples of positioning in the placement guide enclosed.

5) Remove the electrode pads from your skin by holding the pad itself and gently pulling.

6) After use always replace the electrode pads on the plastic liner and replace them in the re-sealable plastic bag.

### Advice

- The electrode pads supplied are reusable but for single patient use.
- The electrode pads provided are latex-free.
- To obtain the best conductivity through the electrode pads always ensure that they are in good condition and tacky.
- Replace the electrode pads when they lose their stickiness or dry out.
- In very hot weather the gel on the electrode pads may become soft.

### Storage life

Electrode pads should last 12 to 20 applications, depending on skin condition and humidity.

Storage life of an unopened pack of electrode pads is 2 years. This may be affected by very high temperatures or very low humidity.

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Fabriqué par:

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TensCare Ltd,  
Manufactured by:  
Tel: +44(0) 1372 723434  
Epsom, Surrey KT19 9BE, UK  
9 Blenheim Road,

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