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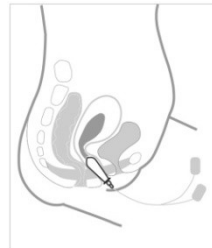
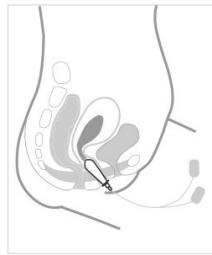
CE
0473

TensCare™

Tenscare Ltd
9 Blenheim Road
Epsom
Surrey
KT19 9BE
Tel +44(0)1372 723434
Fax+44(0)1372 745434
Email: painaway@tenscare.co.uk
www.tenscare.co.uk

Rev 1.1 2/2014

TensCare™

Liberty Plus
Vaginal Electrode
Intravaginal-Elektrode
Sonde vaginale
Espanol

REF X-VPM ∅ 32mm

5 033435 130610

X-VPM Liberty Plus

Indications for Use

1

For use as an intravaginal probe in the *treatment* of female urinary incontinence with muscle stimulation and EMG biofeedback equipment approved to EN60601-01.

IMPORTANT

- For single patient use only
- May be re-used by the same patient
- Wash before first use
- Do not use if pack is open or damaged

Contraindications

Do not use a Muscle Stimulator if the following are present:

- Implanted pacemaker - *May affect correct operation of the device*
- Pregnancy - *Effect on foetal development is not known*
- Bladder or vaginal infection, or recent surgical scar - *Probe insertion may aggravate infection*
- With a history of carcinoma at the site of stimulation - *Laboratory tests show that electrical stimulation may enhance cell division*

Caution is advised

- On patients with altered or reduced tissue sensation - *Excessive stimulation may cause tissue inflammation*
- If the user has a history of epilepsy - *Use of electrical stimulation may lower seizure threshold*
- If the user has a severe prolapse - *Introduction of the probe, or muscle contraction, could damage the prolapsed tissue.*
- Contains nickel:
May cause irritation or allergic reaction following prolonged contact

2

Cleaning

The Liberty electrode is designed for single patient use. Before first use, and after each subsequent use, the unit should be cleaned as follows:
Wash electrode including cable in mild soapy water.
Rinse thoroughly in clean water.
Dry the electrode with a clean cloth/paper towel and allow to air dry before returning to the plastic bag.
DO NOT immerse in liquid.
DO NOT use boiling water.

Disposal

Clean thoroughly and dispose of according to local regulations

Technical Specifications:

Material: Electrode body: GE Plastic MG47

Electrodes: Stainless steel#301

Cables: PVC over copper tinsel conductor

Weight: 20 gm

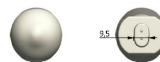
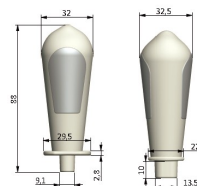
Length: 73 mm (inserted part)

Width: 32mm dia (widest point)

Connections: 2mm pins

Electrode surface area: 7.45cm²

“In Use” product lifetime :6 months


Instructions for Use

3

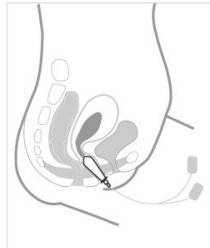1. Remove the electrode from packaging and clean as above.
2. Insert the lead in this pack into the probe.
3. Apply a thin coating of lubricating gel such as TensCare **Go Gel** to the electrode and insert until the flange at the base of the electrode is sited between the labia.
4. Ensure that the muscle stimulator or EMG biofeedback equipment is turned off and connect the unit to the electrode via the connector ends.
5. Follow manufacturer's instructions to operate the stimulator or biofeedback unit.
6. After use, turn off equipment. Disconnect the cable from the electrode and clean as above.

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