

i sock

Part No: K-SOCK

TensCare™

Conductive Garment for use with TensCare TENS & EMS units.

For single patient use only.

Two **MEDIUM** size socks supplied with adapter lead.

Supplied with one pack of four reference electrode pads (50 x 100 mm electrodes).



TensCare Ltd

PainAway House, 9 Blenheim Road,
Longmead Business Park,
Epsom, Surrey KT19 9BE UK.
Tel: +44 (0)13 72 72 34 34
Web: www.tenscare.co.uk



TensCare have altered the specification of the isock to improve performance:-

- We have changed the fabric to a higher silver content to give improved contact.
- We now supply two socks for use with each foot.
- We now supply one pack of four self-adhesive reference electrode pads.

Instructions for Use

1. Ensure that the stimulator is turned **OFF**.
2. Insert one pin connector (from one lead pair) into the connector attached to the isock, and the other (from the same lead pair) into the connector on the reference electrode pad.
3. The isock should be worn slightly damp to improve current flow.
4. Apply the isock and electrode.
5. Place the electrode pad on the same limb as the isock.
6. After use, return the electrode pad to the waxed paper sheet and seal it in the polythene bag supplied.
7. If necessary, hand wash the isock and dry flat at room temperature.

CAUTION

If being used for pain management for Diabetic Neuropathy or other conditions where there is reduced or no sensory sensation then there is a small risk that reduced or uneven electrical contact may cause irritated or broken skin. Care needs to be taken to monitor the skin condition under the garment at frequent intervals.

Pub No: I-SK 10 /13