1 Reusable joint electrode pad for electrotherapy unit

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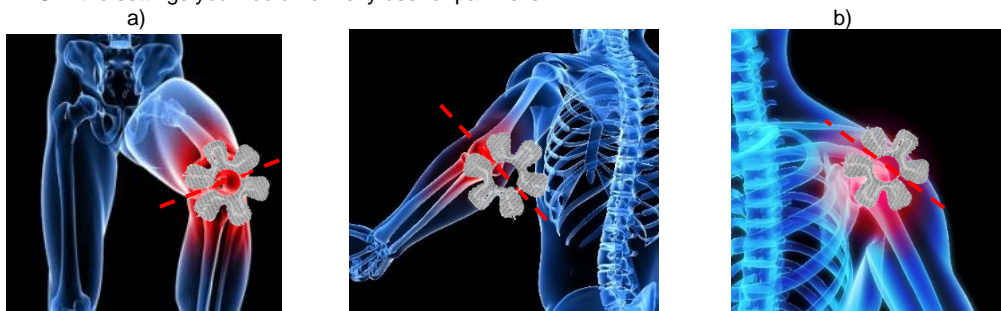
READ CAREFULLY BEFORE USE

Instructions for Use

Use with a TENS unit such as TensCare **perfect TENS** or TensCare **Sports TENS**. Use this electrode pads with TENS in the settings you would normally use for pain relief.

- 1) Before use make sure your skin is clean and dry. You can use conductivity barrier wipes for effective electrode pad contact such as TensCare Contact Skin Wipes (see **E-CW50**).

Do not place electrode pad on skin, which does not have normal sensation or on broken skin.

- 2) Push the pin ends of the lead wires firmly into the pigtail ends of the electrode pad.

Note: The joint electrode pad acts as a pair of TENS electrode pads. Connect both pigtail leads to the same TENS lead coming from the unit.

- 3) Peel the electrode pad from its protective plastic shield by holding and lifting one corner of the pad and pulling.

Do not pull on the pigtail wire of the pad.

Always check that the unit is **OFF** before applying or removing the electrode pad.

- 4) Apply the adhesive side of the electrode pad around the area of pain and firmly press it to the skin.
- 5) Place the electrode pad:
 - a) With dividing line across the joint
 - b) With dividing line along the joint
- 6) Remove the electrode pad from your skin by holding the pad itself and gently pulling.
- 7) After use always replace the electrode pad on the plastic liner and replace them in the re-sealable plastic bag.

Advice

- The electrode pad supplied is reusable but for single patient use.
- The electrode pad provided is latex-free.
- To obtain the best conductivity through the electrode pad always ensure that they are in good condition and tacky.
- Replace the electrode pad when it loses its stickiness or dry out.
- In very hot weather the gel on the electrode pad may become soft.

Storage life

The electrode pad should last 12 to 20 applications, depending on skin condition and humidity. Storage life of an unopened pack of electrode pad is 2 years. This may be affected by very high temperatures or very low humidity.

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TensCare Ltd,